


Designation	MEDIBASE NIAOULI WIPES (100)
Reference	3160746
Product photo	
Product description	Wipes impregnated with a cleaning and disinfectant for surfaces, materials, equipment and furniture which are not used for direct contact with food or feeding stuffs
Characteristics and compositions	<p>Product intended for rapid cleaning and disinfection:</p> <ul style="list-style-type: none"> - Of non-porous hard surfaces in the healthcare sector and community environments (door handles, workstops, furniture, sanitary facilities, etc...) - Of non-porous hard surfaces of non-critical medical devices, as part of the control of microbiological hazards related to the environment (medical beds, monitoring tool, connectors and cables, etc.) <p>Composition :</p> <p>The impregnation solution includes: Disinfectants</p> <ul style="list-style-type: none"> • Didecyltrimethylammonium chloride (n°CAS: 7173-51-5): 0.29% (m/m); • < 5% non-ionic surfactants. • Perfume. <p>Spectre d'action : 5 min 20°C :</p> <ul style="list-style-type: none"> • Bactericidal (EN 16615*, EN 13727*, EN 13697*, EN 1276*); • Yeasticidal (EN 16615*, EN 13624*, EN 13697*, EN 1650*); • Limited virucidal activity (EN 16615*, EN 14476*, EN 16777*). <p>*3g/L bovine albumin + 3mL/L sheep erythrocytes **3g/L bovine albumin</p>
Packaging	Sachet of 100 lingettes
Lifetime	24 month

Storage	Store away from frost and do not exceed 40°C
Indications for use	<p>Instructions for use :</p> <p>Use 4 wipes per m² surface to be cleaned and disinfected; make sure to completely wet the surface. Allow to take effect for 5 minutes. Do not rinse off. Renew the application whenever necessary. Close the packaging well after each use to prevent the wipes from drying out. It is advisable to test the product on a small area to check its compatibility with the surface to be treated. List of compatible materials available on request. Product not incompatible with PC-ABS materials.</p> <p>Precautions for use :</p> <p>Wash hands thoroughly after use. Avoid release to the environment. Dispose of the container and wipes in the appropriate waste collecting sites in accordance with local regulations in force. In the event of an incident, contact the nearest poison control center.</p>
Legal information	Biocide TP2

Version 1.0 – 19/12/2024